

Beltone Direct Line Accessory Pairing: Quick Guide

Before you start:

- Make sure the hearing aids have been programmed and saved in the fitting software
- Batteries are in the hearing aids with the battery doors open
- Make sure the accessories are fully charged
 - myPAL, Remote Control 2 and Phone Link 2 need to charge for at least 3 hours
 - The TV Link 2 needs to be plugged in

Remote Control 2:


- Turn on the Remote Control 2 by sliding the On/Off switch to the down position so the white line is shown
- Remove the silver cover from the bottom of the Remote Control 2
- Push the white pairing button on the front of the Remote Control 2
- Close the battery doors on the hearing aids within 20 seconds of pushing the pairing button on the Remote Control 2
- A melody will play in the hearing aids following a successful pairing



Phone Link 2:

Pairing to the Hearing Aids:

- Turn on the Phone Link 2 by sliding the On/Off switch to the down position so the white line is shown
- Remove the silver cover from the bottom of the Phone Link 2
- Press the **White** pairing button on the back of the Phone Link 2
- Close the battery doors on the hearing aids within 20 seconds of pushing the pairing button
- A melody will play in the hearing aids following a successful pairing

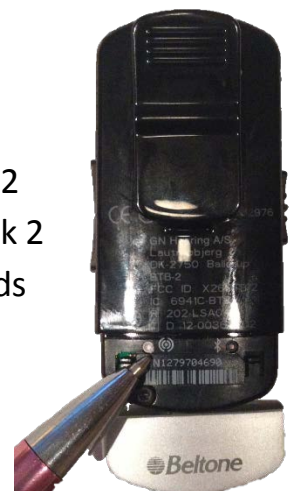

Beltone Direct Line Accessory Pairing: Quick Guide

Pairing to Bluetooth Cell Phone:

- Turn the cell phone On and turn On the Bluetooth function
- Place the cell phone next to the powered On Phone Link 2
- Press the **Blue** Bluetooth pairing button on the back of the Phone Link 2, pairing mode will be enabled for 2 minutes
- On the cell phone select search for Bluetooth devices
- The phone should present a list of discovered Bluetooth devices, select "Hearing Aid Phone" from the list
- If a pass code is required enter 0000
- Pairing to a Bluetooth cell phone will depend on the cell phone manufacturer
- An online Bluetooth pairing guide is available here: www.jabra.com/gnhearing



TV Link 2:

- Make sure the TV Link 2 is plugged in
- Push the pairing button on the back of the TV Link 2
- Close the battery doors on the hearing aids within 20 seconds of pushing the pairing button
- A melody will play in the hearing aids following a successful pairing



myPAL (Original):

- Make sure the myPAL is turned on by sliding the On/Off switch to the down position so the white line is shown
- Remove the silver cover from the myPAL
- Push the (red) pairing button on the front of the myPAL
- Close the battery doors on the hearing aids within 20 seconds of pushing the pairing button on the myPAL
- A melody will play in the hearing aids following a successful pairing



Beltone Direct Line Accessory Pairing: Quick Guide

myPAL Pro and Micro:

- Make sure the myPAL is turned on by pressing the On/Off button on the top of the myPAL for approximately 1.5 seconds until the status LED turns green
- Push the white pairing button on the back of the myPAL
- Close the battery doors on the hearing aids within 20 seconds of pushing the pairing button on the myPAL
- A melody will play in the hearing aids following a successful pairing



*****Note about pairing TV Link 2 and myPAL to the same Hearing Aids*****

- The TV Link 2 and myPAL are both streaming devices and the hearing aids read them the same
- In order to have multiple streaming devices paired to one set of hearing aids you will need to pair the second or third device into streaming program number two or three
- To do this just press the pairing button on the accessory twice to pair into program number two or thrice to pair into program number three